

ReNu MultiPlus® Multi-Purpose Solution

This insert contains important information. Please read it carefully and retain for future reference.

Bausch & Lomb ReNu MultiPlus Multi Purpose Solution

With HYDRANATE® Protein Remover

Fresh Lens Comfort

Moisturises, Cleans, Disinfects, Lubricates, Rinses, Stores and Removes Protein.

CONTENTS: A sterile, isotonic solution that contains boric acid, edetate disodium, sodium borate and sodium chloride; DYMED® (polyaminopropyl biguanide) 0.0001%, HYDRANATE® (hydroxyalkylphosphonate) 0.03%, poloxamine 1%.

ACTIONS: Cleans, loosens and removes accumulations of film, protein and other deposits from soft contact lenses. Destroys harmful micro-organisms on the surface of the lens. Rinses, stores and lubricates.

INDICATIONS: BAUSCH & LOMB ReNu MultiPlus® Multi-Purpose Solution is indicated for use in chemical (not heat) disinfection, daily cleaning, daily protein removal, lubricating, rinsing and storage of all soft contact lenses as recommended by your eye care practitioner.

DIRECTIONS: Always wash and rinse your hands before you handle your lenses. Lenses must be thoroughly cleaned, rinsed and soaked each time they are removed to achieve disinfection.

To clean, disinfect and remove protein daily, complete these simple steps. This daily regimen is recommended by Bausch & Lomb for a healthy and comfortable contact lens wearing experience:

Step 1: Place 3 drops of ReNu MultiPlus Multi-Purpose Solution on each side of the lens surface and gently rub for 10 seconds on each side.

Step 2: Thoroughly rinse each side of the lens for 5 seconds with ReNu MultiPlus Multi-Purpose Solution.

Step 3: Place cleaned contact lens in the lens case and fill with fresh ReNu MultiPlus Multi-Purpose Solution. Soak at least 4 hours. Remember to always use fresh solution – never reuse solution.

Always follow your eye care professional's instructions. Based upon your individual tear chemistry and lens-wearing schedule, your eye care professional may recommend additional products or procedures. Your eye care professional may also recommend an alternative procedure without a rubbing step. If so, to clean, disinfect and remove protein daily without a rub step, complete Step 2 and Step 3 only, as described above.

If any debris remains on your lenses you may rinse with ReNu MultiPlus Multi-Purpose Solution. Pour the BAUSCH & LOMB ReNu MultiPlus Multi-Purpose Solution out of the lens case and rinse the case with fresh ReNu MultiPlus Multi-Purpose Solution. Leave the case with the caps off to dry in the air. Replace the caps. Your case is now ready for use.